


## User Instruction for the Deluxe Percher – Steel Arms & Steel Back C279

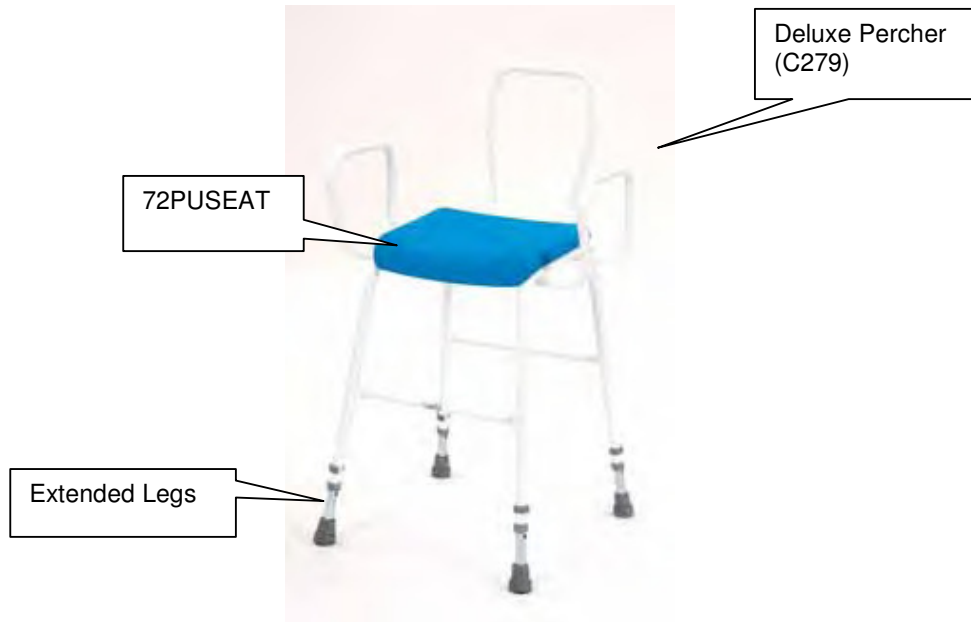

### 1. Introduction

This Deluxe version of the Perching Stool Range features deeply contoured polyurethane foam seats with pronounced mouldings to counter any slippage from the inclined seat. The recently improved framework now caters for weight up to 157kg. The variable height options cater for the majority of users and the splayed legs of a frame give maximum frame stability.

Product Dimensions	
Seat height (from-to)	575mm-725mm
Width between arms	480mm
Seat to top of arms	150mm
Seat width	355mm
Seat depth	270mm
Seat incline	25mm
Footprint	480mm x 620mm

### 2. Contents of the Box

- Deluxe Percher (C279)
- Seat (72PUSEAT)
- Extended Legs & Pin Clips
- User Instructions (this leaflet)

### 3. Assembly Instructions

- Carefully unpack and examine the product to ensure it is complete.
- Before use check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product. **DO NOT ATTEMPT TO USE A DAMAGED PERCHER.**

### 4. Correct Use

Adjustable Height:

## User Instruction for the Deluxe Percher – Steel Arms & Steel Back

### C279

- Remove the pin clips from the legs.
- Select the desired height of the frame by gently sliding out each extension leg from the frame - this is a telescopic action.
- When the desired height is obtained refit the pin clips into the appropriate hole ensuring each pin clip goes through each leg and its extension leg and that all legs are set to the same height.
- The stool is now ready for use.

#### 5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this will damage the frame and nullify your warranty.
- It is not recommended that polyurethane is cleaned by the use of power/jet washing or industrial washing machines.

#### 6. Maintenance Requirements

The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

#### 7. Warning

- This product is intended for users of a maximum of 157 kg/25 stone in weight.
- We recommend a full risk assessment should be carried out before use.

#### 8. Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

#### 9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non hazardous and can be disposed of appropriately.

#### 10. Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare  
Cefndy Road  
Rhyl  
Denbighshire  
LL18 2HG  
Tel: 0044 1745 343877 (Quality Assurance Dept)  
E-mail: [cefndyengineering@denbighshire.gov.uk](mailto:cefndyengineering@denbighshire.gov.uk)

