

User Instruction for the Rise & Fall Table Range B43 Rise & Fall Table & B431 Low Height Version

B430 with Wheelchair Friendly Base & B432 Low Height Version

1. Introduction

The Rise & Fall Table is designed to aid the user for dining, reading or writing. This table offers luxury for both the user and the carer. Its spring-loaded easy rise and fall mechanism enables clients with poor grip to adjust by finger action only. It has a large table surface area and is adjustable in height. The beech polished top is hard wearing and easy to clean, and the retaining table edges prevent items slipping off. The B430 and B432 models have a U-Shaped frame that enables the products to be used over a bed or around a wheelchair.



Product specification	
Rise (from-to) B43 & B430	750mm – 1100mm
Rise (from-to) B431 & B432	630mm – 825mm
Height of legs from floor	90mm
Top size	810mm x 455mm
Top size B430	780mm x 1114mm
Footprint B43 & B431	810mm x 455mm
Footprint B430 & B432	970mm x 440mm
Parts list	
Description	Part No
Castors (Set of 4)	700231/1

2. Contents

- User Instructions (this leaflet)

3. Assembly Instructions

- Carefully unpack and examine the product to ensure that it is complete.
- Before use, check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product. **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**

4. Correct Use

- The table is designed to help the user by being used as a dining, reading, or writing aid.
- The table is not intended to support full body weight so refrain from attempting to lift full body weight via the product
- The table should be placed either against a bed or chair.
- Do not attempt to overload the table.



Height Adjustment:

- To adjust the height of the table top, press and hold the locking bar situated on the side of the table, the table top will now freely move.
- When the desired height is achieved release the locking bar, the table top will now lock into position.

To lower the table top height press and hold in the locking bar and press down on table top (at the column end) to the desired height and then release the locking bar.



User Instruction for the Rise & Fall Table Range B43 Rise & Fall Table & B431 Low Height Version

B430 with Wheelchair Friendly Base & B432 Low Height Version

5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this may damage the frame and nullify your warranty.
- Do not clean this product by means of power/jet washing and industrial washing machines.

6. Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

7. Warning

- This product is intended for use as a dining, reading or writing aid – it does not support heavy weights.
- We recommend a full risk assessment should be carried out before use.

8. Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non hazardous and can be disposed of appropriately.

10. Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 0044 1745 343877 (Quality Assurance Dept)
E-mail: cefndy.sales@denbighshire.gov.uk
