


User Instruction for the Mobile Shower Chair Range – S33/2014 with Fixed Height & S34/2014 with Adjustable Height

1. Introduction

The S33 and S34 have swivel castors to provide easy manoeuvrability and a brake on each castor for added user safety. The moulded plastic seat has drainage holes which conform to MHRA guidelines and the cross bracing provides additional product stability.

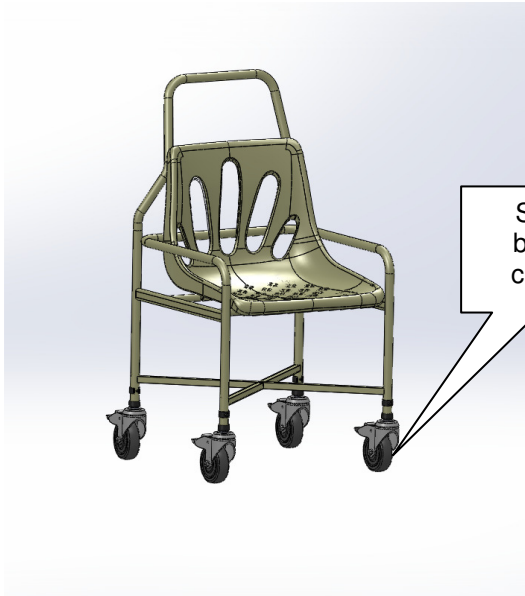

Product specification	
Seat height (S33)	450mm 17 ½ ”
Seat height (S34)	440mm – 590mm 17 ¼ ” – 23 ¼ ”
Seat depth	370mm 14 ½ ”
Seat to top of arms	110mm 4 ¼ ”
Seat to top of back	350mm 13 ¾ ”
Width between arms	450mm 17 ¾ ”
Footprint Front to back	570mm 23 ½ ”
Footprint Side to side	535mm 21”
Maximum user weight S33 (fixed height)	159 KG (25 Stone)
S34 (adjustable height)	127 KG (20 Stone)
Parts list	
Description	Part No
4” braked castor	7004TL (S33 only)
4” braked castor	700S34 (S34 only)
Push on pin clips	63PC
Anti-rattle collars	74GAR/2
Seat	72031/62-2014

2. Contents

- Mobile Shower Chair
- User Instructions (this leaflet)
- Batch label – This will be found on the product and will include the product code as well as a number. In the event of needing to contact the manufacturer about this product always quote the Batch Number.

3. Assembly Instructions

- Carefully unpack and examine the product to ensure that it is complete.
- Before use check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product.
- **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**

4. Correct Use

Height Adjustment: (S34 only)

- Remove the pin clip from each of the legs.
- Select the required height of the frame by gently sliding each extension leg in or out.
- When the desired height is found refit the pin clip into the appropriate hole in each extension leg.



User Instruction for the Mobile Shower Chair Range – S33/2014 with Fixed Height & S34/2014 with Adjustable Height

- Ensuring that it goes fully through, ensure that all legs are set to the same height.

5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis and more frequently if required.
- Do not use an abrasive cleaner as this will damage the frame and nullify your warranty.
- This product is not suitable to be cleaned using power/jet washing and industrial washing machines.

6. Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

7. Warning

- The S33 is intended for users of a maximum of 159kg/25stone in weight.
- The S34 is intended for users of a maximum of 127kg/20stone in weight.
- We recommend a full risk assessment should be carried out before use.
- This product has been designed for use on flat surfaces only **do not use over thresholds with a user in situ**
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

8. Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of appropriately.

10. Contact Details

We have attempted to cover most eventualities; however, if you have any specific queries please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 0044 1745 343877 (Quality Assurance Dept)
E-mail: cefndy.sales@denbighshire.gov.uk
