

User Instruction for the stackable/adjustable height – Raised Toilet Seats, models; T80, T81 (with lid) & T8080 (with floor fixing feet) – 2013 versions


1. Introduction

The Raised Toilet Seat frame is designed to enhance user comfort with an oval seat. The product is adjustable in height from 450mm – 600mm at 25mm intervals. The feet (standard) are none-marking, none-slip for extra user safety.

All structural joints are fully welded and the toilet seat clips on and off. These features are designed to aid product hygiene by reducing the possibility of bacterial entrapment. The steel framework is also coated in an anti-bacterial powder coated finish. The clip-on design toilet seat aids maintenance as no tools are required to change a seat.



Product specification	
Toilet seat height	450mm – 600mm
Max width between arms	485mm
Toilet seat depth	420mm
Seat to top of arms	150mm
Footprint min @ 450mm Ht	550mm x 576.5mm
Footprint min @ 600mm Ht	550mm x 630mm
Maximum user weight	159 KG (25 Stone)

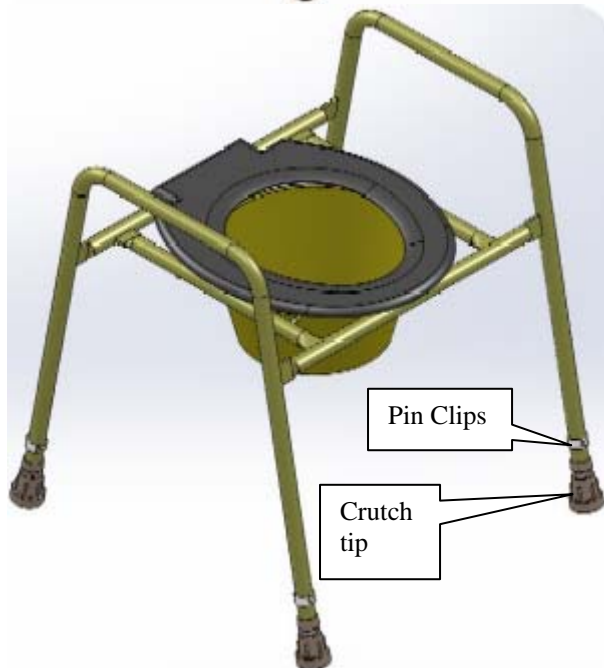

Parts list	
Description	Part No
Toilet seat & Lid (T81 only)	71LBS & 71LBL
Crutch tip	74G3
Extension leg	56C319P
Pin clip	63PC
Crutch tip	74GAR/2

Optional Parts list	
Description	Part No
Sleeve (optional)	T171
Potty	T173
Floor fixing feet	T174

User Instruction for the stackable/adjustable height – Raised Toilet Seats, models; T80, T81 (with lid) & T8080 (with floor fixing feet) – 2013 versions

2. Contents

- Raised Toilet Seat adjustable height– White (T80, T81 or T8080)
- User Instructions (this leaflet)
- Batch label – This will be found on the product and will include the product code as well as a number. In the event of needing to contact the manufacturer about this product always quote the Batch Number.

3. Assembly Instructions

- Carefully unpack and examine the product to ensure it is complete.
- Before use, check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product. **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**

4. Correct Use

This product has been designed specifically to aid toileting requirements either over an existing system or as a stand alone item fitted with a potty.

- Ensure legs are all set to the same height (see height adjustment)
- If optional floor fixed feet are used ensure they are adequately secured to an appropriate surface
- Please empty the (optional) potty at regular intervals and be aware of its maximum capacity.
- **Do not attempt to move the Toilet Seat whilst occupied.**

T171 or T173- Sleeve or potty (optional):

- To locate the optional sleeve or potty in the Raised Toilet Seat, raise the toilet seat to a vertical position.
- By holding the sleeve or potty place on the support brackets ensuring an accurate location of the wide brim on the brackets.
- Remove lid (potty only), fold the potty handle down and lower the toilet seat into the horizontal position.

Removing the Sleeve / Potty:

Once the commode has been used, repeat the process in the reverse order ensuring that the lid (potty only) is correctly placed to avoid possible spillage.

T174 - Floor Fixing Feet (optional, standard with T8080) for added location security:



- Remove the pin clips from each of the 4 legs and remove the extension legs completely.
- Fit the 4 Floor Fixing Feet to the product and replace the pin clips - ensure the Fixing feet are set at the same height.
- Position the product exactly where required, and ensuring it is not moved, mark out the hole positions for each of the Floor Fixing Feet.
- Remove the product and using a drill bit of the type and size appropriate to the underlying floor material and whether plastic inserts are required. Drill each hole to the depth necessary dependent upon the use of plastic inserts and type of screws to be used. - If in doubt seek advice from professionals
- Using screws of the appropriate type and size, secure the product to the floor ensuring screws are fitted to all of the 12 holes in the Floor Fixing Feet.
- Remove any debris.

User Instruction for the stackable/adjustable height – Raised Toilet Seats, models; T80, T81 (with lid) & T8080 (with floor fixing feet) – 2013 versions

Height Adjustment:



- Remove the pin clip from each of the legs
- Select the required height of the frame by gently sliding each extension leg in or out.
- When desired height is found refit the pin clip into the appropriate hole in each extension leg
- Ensuring it goes fully through. Ensure that all legs are set to the same height.

5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- The toilet seat and retaining clips can be removed to aid cleaning. **This feature is an added benefit over traditionally fixed toilet seats as it ensures all surfaces are fully accessible for cleaning.**
- Do not use an abrasive cleaner as this may damage the frame and nullify your warranty.
- It is recommended that the potty is thoroughly cleaned after every use, using an appropriate cleaning agent.
- Remove legs for cleaning.

Use of power/jet washing & industrial washing machines:

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal moisture evaporates).
- We cannot offer advice on industrial washing machines with regards to specific machine settings, as these will be dependant on the manufacturer's instructions.

6. Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

7. Warning

- This product is intended for users of a maximum of 159kg/25stone in weight.
- We recommend a full risk assessment should be carried out before use.
- Under no circumstance should the commode be moved whilst occupied.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

8. Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage **however we can advise that this product has been designed, manufactured and independently tested in the UK (including cyclic testing to represent real life usage).**



**User Instruction for the stackable/adjustable height –
Raised Toilet Seats, models; T80, T81 (with lid) & T8080
(with floor fixing feet) – 2013 versions**

9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non hazardous and can be disposed of appropriately.

10. Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 0044 1745 343877 (Quality Assurance Dept)
E-mail: cefndy.sales@denbighshire.gov.uk
