

This product can be purchased in various configurations as detailed in the table. There are also 3 options in width between the arms with are identified by the prefix in italics at the beginning of the product code, e.g. X161/15/V.

Mediatric™ – 600mm

(product code begins with “M”)

Bariatric – 700mm

(product code begins with “X”)

Super Bariatric – 800mm

(product code begins with “SX”)

Product code & description	
158/15/PU	Basic (no arms) – Polyurethane seat (PU)
158/15/V	Basic (no arms) – Vinyl seat
159/15/PU	Steel Arms – PU seat
159/15/V	Steel Arms – Vinyl seat
160/15/FB/PU	Steel Arms & Back – PU seat
160/15/FB/V	Steel Arms & Back – Vinyl seat
161/15/DB/PU	Steel Arms & Padded Detachable Back – PU upholstery
161/15/DB/V	Steel Arms & Padded Detachable Back – Vinyl upholstery
161/15/FB/PU	Steel Arms & Padded fixed Back – PU upholstery
161/15/FB/V	Steel Arms & Padded fixed Back – Vinyl upholstery

Seat height range = 450mm – 600mm (measured from the front of the seat)

1. Introduction

The Perching Stool Range is designed to enhance user comfort with heat sealed padded vinyl upholstery or polyurethane foam. The Perching Stool is adjustable in height at 25mm intervals. The feet are non-marking & non-slip for extra user safety.

All structural joints are fully welded. As well as providing structural strength this feature also aids product hygiene by reducing the possibility of bacterial entrapment. The steel framework is also coated in an anti-bacterial powder coated finish.



Version pictured is an M160/15/FB/PU

Product specification	
Seat depth	320 mm
Max footprint “M” range	660mm x 560mm
Max footprint “X” range	760mm x 560mm
Max footprint “SX” range	860mm x 560mm
Seat to top of arms	160mm
Seat to top of back	390mm
Maximum user weight	318 KG (50 Stone)

Parts list	
Description	Part No
Crutch tip	74G3
Extension leg	56C319P
Pin clip	63PC
Anti-rattle collar	74GAR/2
Hand wheel (DB only)	73046
PU Back (all)	72PU600BACK
PU Seat (all)	72PU600SEAT
Vinyl Back	67057BEW/G/2
Vinyl seat “M” range	67057SEW/G/2
Vinyl seat “X & SX” range	67YESS/SEAT

2. Contents

- Perching Stool
- User Instructions (this leaflet)
- Batch label – This will be found on the product and will include the product code as well as a number. In the event of needing to contact the manufacturer about this product always quote the Batch Number.

3. Assembly Instructions

- Carefully unpack and examine the product to ensure it is complete.
- Before use check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product.
- **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**

4. Correct Use –

- **To adjust the Perching Stool height**

1. Remove the pin clip from the leg individually
2. Select the height by positioning each extension leg individually aligning the holes in the extension leg and frame
3. When the required height is selected refit the pin clip into the appropriate hole
4. Repeat the above on the other three legs
5. Ensuring all four pin clips are fully engaged and check that all four legs are set to the same height
6. This product is designed so the seat will be lower at the front than the rear (see warning section)



- **To adjust the removable back (DB versions only)**

1. Unscrew the handle wheels (clockwise) around half one full turn
2. The detachable back should now slide freely in the locating brackets
3. Once the desired position is reached retighten (anti-clockwise) the four handle wheels to lock the back in position
4. As an added safety feature pin clips are inserted between the hanging brackets to stop the back from sliding out of the brackets **ensure these pin clips are always inserted when the product is in use**



The back may have been located in an inverted position for transit. If so remove the two safety pin clips located on the back between the hanging brackets, slightly unscrew the hand wheels as above, fully withdraw the back from the brackets. Then rotate the back to an upright position and sliding the back into all four brackets, insert the safety pin clips & repeat the adjustment process to locate and secure in the back in correct position.

5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this will damage the frame and nullify your warranty.
- It is not recommended that polyurethane is cleaned using power/jet washing or industrial washing machines.

Use of power/jet washing & industrial washing machines (Not recommended for PU products):

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal moisture evaporates).
- We cannot offer advice on industrial washing machine

6. Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

7. Warning

- **This product is designed so when in use the seat will be lower at the front than the rear, the easy way to check if the product is set for correct use is to ensure that you can see an equal number of holes visible on all four legs. The product may have been sent with the front legs lower than the rear for transportation purposes.**
- This product is intended for users up to a maximum of 318kg/50stone in weight.
- We recommend a full risk assessment be carried out before use.
- Under no circumstance should the Perching Stool be moved whilst occupied.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

8. Warranty

This product carries a 12 month warranty from the date of purchase from Cefndy against manufacturing defects. Misuse or accidental damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage **however we can advise that this product has been designed, manufactured and independently tested in the UK (including cyclic testing to represent real life usage).**



**User Instruction for the 2015
Mediatric/Bariatric & Super
Bariatric Perching Stool Range**

9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of appropriately.

10. Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 0044 1745 343877 (Quality Assurance Dept)
E-mail: cefndy.sales@denbighshire.gov.uk
