

User Instruction for the High Back Chair – Winged & Moulded Arms (Brown)
C131



1. Introduction

Your High Back Chair is made with easy clean upholstery for longevity of service. The tubular metal framework is coated in high gloss nylon epoxy powder coating. The chair is adjustable in height and lumbar support is given with the profiled back. The framework design and the none-slip rubber feet both work towards patient stability. This version offers side head supports with the winged back.

Product Dimensions	
Seat height	470mm x 572mm
Seat to top of back	698mm
Seat to top of arms	184mm
Width between arms	482mm
Footprint	762mm x 609mm

2. Contents of the Box

- High Back Chair (C131)
- Winged Back (67WSFBW)
- Moulded Arm (67CELARM)
- Seat (67SFS)
- Extended Legs & Pin Clips
- User Instructions (this leaflet)

3. Assembly Instructions

User Instruction for the High Back Chair – Winged & Moulded Arms (Brown)

C131

- Carefully unpack and examine the product to ensure it is complete.
- Before use check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product. **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**

4. Correct Use

Adjustable Height:

- Remove the pin clips from the legs.
- Select the desired height of the frame by gently sliding out each extension leg from the frame - this is a telescopic action.
- When the desired height is obtained refit the pin clips into the appropriate hole ensuring each pin clip goes through each leg and its extension leg and that all legs are set to the same height.
- The chair is now ready for use.

5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this will damage the frame and nullify your warranty.
- This product is **not** suitable for power/jet washing or industrial washing machines.

6. Maintenance Requirements

The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

7. Warning

- This product is intended for users of a maximum of 157kg/25stone in weight.
- We recommend a full risk assessment should be carried out before use.

8. Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non hazardous and can be disposed of appropriately.

10. Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 0044 1745 343877 (Quality Assurance Dept)
E-mail: cefndyengineering@denbighshire.gov.uk

